

## Liquid Dam 17-313-201

### INSTRUCTIONS FOR USE:

#### RECOMMENDED INDICATIONS:

1. Is a simple, effective, and time-saving approach to saliva isolation and soft or hard tissue protection. This low viscosity, blue liquid flows evenly and is easily distinguished from teeth and gingiva. Peel away the rubberized resin film and discard when restoration is complete.
2. Is also excellent for repairing traditional dental dam.

#### CONTRAINDICATIONS:

In rare cases the product may cause sensitivity in some people. If any such reactions occur, discontinue the use of the product and refer to a physician.

#### COMPOSITION:

Methacrylate

#### PRODUCT PRESENTATION:

3ml Syringe # 17-313-201

#### INSTRUCTIONS FOR USE:

1. Remove black cap (do not discard) and dispense small amount of **Dental Dam** material on a paper pad. Then apply a bead of dental dam on gingival surface surrounding dentition to be isolated.
2. Carefully manipulate to thin film with the syringe tip or spread with a sable brush. Clean brush and/or syringe tip in organic solvent before ambient light causes polymerization. Replace cap on syringe.
3. Holding light tip 1/2 inch (1.3cm) away from **Dental Dam** material, light cure 45-60 seconds with standard dental curing lamp. If using a plasma arc light, 15-20 seconds curing is sufficient. Repeat until the entire surface has been cured.
4. Peel away with explorer or similar instrument when procedure is completed


#### WARNING:

1. Medical devices should be kept out of reach of children.
2. Make sure to replace caps on syringes immediately after use to prevent premature curing from ambient light sources.

#### USE AND STORAGE:



Store away from sunlight

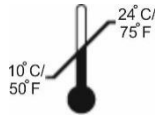

Observe storage temperatures



Protect from moisture



Shelf life, 2 years

\* The product can be refrigerated to extend shelf life. If material is refrigerated, allow material to warm to room temperature before using; 72°F (22°C).

#### EMERGENCY MEASURES:

Eye contact – Flush with copious amounts of water. Seek medical attention if irritation persists.

Ingestion - Dilute by drinking water.

Skin contact – Wash with soap and water.

CAUTION: Federal law restricts this device to sale by, or on the order of, a dentist (or dental professional).