

17-912-102

INSTRUCTIONS FOR USE:

RECOMMENDED INDICATIONS:

DuraBond is a simple, moist bonding adhesive containing 10%, 5nm colloidal filler used for:

1. All direct, light-cured bonding procedures.
2. Indirect bonding procedures when combined with RelyX ARC Adhesive Resin Cement.
3. Root surface desensitization.
4. Porcelain and composite repair.
5. Bonding veneers in combination with RelyX Veneer Cement

CONTRAINDICATIONS:

This product contains substances that may cause allergic reaction by skin contact in certain individuals.

COMPOSITION:

Ethyl alcohol.
Bisphenol a diglyceryl ether dimethacrylate.
Silane treated silica
Hydroxyethyl methacrylate

PRODUCT PRESENTATION

6 gm / bottle # 17-912-102
3 gm / bottle # 17-912-102-x

INSTRUCTIONS FOR USE:

Isolation: liquid dam (17-313-201) is the preferred method of isolation.

Etching: Apply etchant to enamel and dentin. Wait 15 seconds. Rinse for 10 seconds. Blot excess water using a cotton pellet or mini-sponge. The surface should appear glistening without pooling of water.

Adhesive: Immediately after blotting, apply 2-3 consecutive coats of adhesive to etched enamel and dentin for 15 seconds with gentle agitation using a fully saturated applicator. Gently air thin for five seconds to evaporate solvents. Light cure for 10 seconds.

WARNING:

This product contains substances that may cause allergic reaction by skin contact in certain individuals.


Avoid use this product in patients with known acrylate allergies.

This product is a flammable material.

USE AND STORAGE:



Store away from sunlight

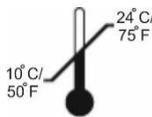

Observe storage temperatures



Protect from moisture



Shelf life, 3 years

- DuraBond should be capped immediately after use to minimize evaporation.

EMERGENCY MEASURES:

Eye Contact: Flush with copious amounts of water. Seek medical attention if irritation persists.

Ingestion: Rinse mouth. If feel unwell, seek medical attention.

Skin Contact: Wash with soap and water.

Inhalation: Remove person to fresh air. If feel unwell, seek medical attention.

Caution: Federal law restricts this device to sale by or on the order of a dentist (or dental professional).

International Orthodontic Services
12811 Capricorn St. Stafford, TX 77477 USA.
T: +1 (832) 342 9487 F: +1 (281) 340 4794
www.iosortho.com info@iosortho.com